

### **Adverse Reactions:**

Allergic or pre-existing skin, eye and respiratory conditions may be aggravated by exposure to this product.

### **Instructions for use:**

1. Prepare and cleanse the root canal in conventional manner.
2. Remove the cap from the syringe and save. Attach a new applicator tip.
3. Prior to application dispense a small amount of paste. This will equalize the pressure in the syringe and control the flow of paste through the applicator tip.
4. Gradually fill the root canal with **Alpha-Cal** while® intermittently compressing with absorbing material such as cotton pellets or paper points. For very narrow root canals it may be necessary to use a Lentulo spiral to distribute paste.
5. Cover with a cotton pellet then seal with your preferred temporary cement.
6. **Alpha-Cal**® is easily removed with file and irrigation.
7. It may be necessary to change the dressing every month in cases requiring prolonged root canal therapy.

### **Storage:**

1. Store material between 10°C/50°F to 27°C/80°F. See outer package for expiration date.
2. Avoid direct exposure to sunlight.
3. Keep syringe capped when not in use.