



ALPHA-EASE®

DESENSITIZER



Description:

Alpha-Ease® Desensitizer is a simple one-step product for treating and preventing dentinal hypersensitivity. It works quickly and effectively to seal the dentinal tubules and inhibit bacterial growth.

Indications for Use:

- Under any indirect resin-bonded inlay, onlay, crown or veneer.
- Under amalgam and composite restorations.
- In conjunction with other dentin bonding adhesives.

Contraindications:

None known.

Warnings:



- Always replace the cap after each use.
- Avoid contact of **Alpha-Ease®** Desensitizer with soft tissue, eyes, and skin. In case of contact with soft tissue, eyes or skin immediately flush with water. If accidental eye contact occurs, immediately flush with water and seek medical advice. Extended contact with tissue can result in redness or burning.

General Precautions:

1. U.S. Federal law restricts this device to sale by or on the order of a dentist.
2. Review instructions and MSDS prior to beginning treatment. Use only as directed.

Adverse Reactions:

Pre-existing skin and eye conditions may be aggravated by exposure to this product.

Instructions for use:

1. Prepare and clean dentin surfaces. It is not necessary that the dentin surface is completely dry, a slight moist surface is acceptable.
2. Apply **Alpha-Ease®** Desensitizer with a brush or any other suitable instrument. A gentle rubbing motion for 10 seconds will assist the delivery of the desensitizer into the dentinal tubules.
3. Allow desensitizer to remain for 30 seconds.
4. Carefully dispense excess to a thin layer with blown air. Dry with an air syringe. Do not rinse.
5. Repeat application if sensitivity recurs or persists.

Storage:

Refrigeration is recommended. Store material between 2°C/36°F to 27°C/80°F. Always bring to room temperature before using (22°C/72°F). Do not freeze. See outer package for expiration date.

DENTAL
TECHNOLOGIES

6901 N. HAMLIN AVENUE
LINCOLNWOOD, ILLINOIS 60712 USA

www.dentaltech.com



EMERGO EUROPE
Molenstraat 15
2513 BH, The Hague
The Netherlands

PM-3032/REV005