ES

**Descripción:**

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EN

**Description:**

**Alpha-Pro**® White Varnish contains 5% sodium fluoride. It is used in the treatment of dentinal and postoperative sensitivity. **Alpha-Pro**® White Varnish goes on clear, unlike traditional varnishes that cause unsightly yellow coloring of the teeth. **Alpha-Pro**® White Varnish sets on contact with saliva, leaving a smooth, thin layer. **Alpha-Pro**® White Varnish is certified Peanut, Tree nut, and Gluten free. 0.50 ml of **Alpha-Pro**® White Varnish contains 25 mg sodium fluoride, equivalent to 11.3 mg fluoride ion.

**Indications for Use:**

For treatment of dentinal and postoperative sensitivity.

**Contraindications:**

**Alpha-Pro**® White Varnish is contraindicated in patients with a history of severe allergic reaction to fluoride and in patients with ulcerative gingivitis and stomatitis.

**Warnings:**

- Do not apply other fluoride preparations, such as fluoride gels and foams, on the same day as **Alpha-Pro**® White Varnish. The use of fluoride tablets should be suspended for 2-3 days after treatment.
- If pregnant, consult a Dentist or Doctor prior to use of **Alpha-Pro**® White Varnish.

**General Precautions:**

- U. S. Federal Law restricts this device to sale by or on the order of a dentist.
- Review instructions and SDS ([www.dentaltech.com/resource](http://www.dentaltech.com/resource)) prior to beginning treatment. Use only as directed.
- The unit dose and brush are disposable and intended for single patient use.

**Adverse Reaction:**

In rare cases, edematous swellings have been reported while using fluoride varnish products. In extremely rare cases dyspnea has occurred in asthmatic individuals after application of some fluoride varnishes. Individuals who are hypersensitive to fluoride may experience nausea on occasion. If necessary, the varnish can easily be removed with a thorough tooth brushing.

**Instructions for Use:**

- Remove excess moisture with air, a cotton swab or roll.
- Carefully open the lid from the unit-dose package. Lightly stir varnish with the provided applicator brush.
- Apply a thin layer of **Alpha-Pro**® White Varnish on the teeth to be treated with the brush. Avoid contact with gums.
- Alpha-Pro**® White Varnish sets on contact with saliva; the patient may leave immediately after application of the varnish.
- Instruct the patient not to brush or floss for at least 2 hours after treatment, and to eat only soft foods immediately after treatment. Beverages, mouthwashes or rinses containing alcohol should also be avoided for at least 2 hours after application.

**Storage:**

- Do not refrigerate. Store material between 20°C/68°F to 25°C/77°F. See outer package for expiry date.
- Do not expose to elevated temperatures or intense light.

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