ES

Descrizione:

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EN

Description:

Alpha-dent® Band Adhesive is a light cured, single paste resin-based, fluoride releasing band adhesive. **Alpha-dent®** Band Adhesive requires no mixing; resulting in reducing adhesive waste, consistent adhesive performance and no working time constraints. **Alpha-dent®** Band Adhesive's distinctive blue color allows you to easily see the adhesive during banding, making clean up easy.

Indications for Use:

Alpha-dent® Band Adhesive is a light cured orthodontic adhesive designed for banding orthodontic appliances to teeth.

Contraindications:

Alpha-dent® Band Adhesive is contraindicated in patients with a history of severe allergic reaction to methacrylate resins.

General Precautions:

- U.S. Federal Law restricts this device to sale by or on the order of a dentist.
- Review instructions and SDS (www.dentaltech.com/resource) prior to beginning treatment. Use only as directed.
- Do not use in the presence of eugenol based materials as complete setting may not occur.
- Do not expose to elevated temperatures or intense light.
- Provide suitable moisture control to isolate area and to prevent moisture contamination during setting.

Adverse Reaction:

Allergic or pre-existing skin, eye and respiratory conditions may be aggravated by exposure to this product.

Instructions for Use:**Tooth Preparation**

- Thoroughly clean teeth to be banded with **Alpha-Pro®** No Added Color/Flavor/Fluoride Proply Paste. Proply paste which contains oil or fluoride should not be used as it may inhibit the banding.
 - Rinse teeth thoroughly with water; dry with oil and moisture-free compressed air.
- Band Adhesive Application**
- For proper fit, trial fit each band on the patient. After fit has been verified, remove band and clean.
 - Hold the band between the thumb and forefinger. Apply a bead of **Alpha-dent®** Band Adhesive to the inside of the band. Cover the entire inside of the band by spreading the paste with a small spatula or other suitable instrument.
 - Set band on tooth, the excess **Alpha-dent®** Band Adhesive can be removed with an explorer or plastic instrument.
 - Using a light-curing unit (minimum output of 600 mW/cm²) cure the material for at least 20 seconds. The light-curing unit tip should be placed in close proximity to the occlusal surface of the tooth being banded.
 - After treatment is complete, remove bands in the usual manner.

Storage:

- Recommended storage conditions are between 2°C/36°F to 27°C/80°F, bringing to room temperature before using (23°C/73°F). Do not freeze. See outer package for expiry date.
- Do not expose to elevated temperatures or intense light.
- Keep container closed when not in use.

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