ES

**Descrpción:**

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EN

**Description:**

**Alpha-Cal®** is a calcium hydroxide paste with barium sulfate used as a root canal sealer for the temporary filling of root canals after conventional endodontic treatments. **Alpha-Cal®** is an aqueous, radiopaque, high pH (>12.0) paste packaged in a syringe ready to be dispensed through disposable tips into the root canal.

**Indications for use:**

- A temporary root canal filling after conventional endodontic treatment.
- Direct pulp capping
- A strong antibacterial agent due to its high pH.
- Stimulates the formation of dentin bridges and hard tissue barrier in roots.

**Warning:**

- **Alpha-Cal®** is corrosive and may dry the skin, causing irritation and burns. Avoid contact with skin and eyes. If accidental contact occurs, immediately flush with plenty of water. In case of contact with eyes, seek medical attention.

**General Precautions:**

- U.S. Federal Law restricts this device to sale by or on the order of a dentist.
- Review instructions and SDS ([www.dentaltech.com/resource](http://www.dentaltech.com/resource)) prior to beginning treatment. Use only as directed.
- **Alpha-Cal®** is supplied within a syringe and is intended to be dispensed with the supplied single use applicator tips.
- Always replace cap after each use.

**Adverse Reaction:**

Allergic or pre-existing skin, eye and respiratory conditions may be aggravated by exposure to this product.

**Instructions for use:**

1. Prepare and cleanse the root canal in the conventional manner.
2. Remove the cap from the syringe and save. Attach a new applicator tip that is provided with the kit.
3. Prior to application, dispense a small amount of **Alpha-Cal®** onto a dispensing pad or other suitable surface to verify the flow, and that the applicator tip is properly attached prior to intraoral use.
4. Gradually fill the root canal with **Alpha-Cal®** while intermittently compressing with absorbing material such as cotton pellets or paper points. For very narrow root canals it may be necessary to use a Lentulo spiral to distribute paste.
5. Cover with a cotton pellet then seal with your preferred temporary cement.
6. **Alpha-Cal®** is easily removed with file and irrigation.
7. It may be necessary to change the dressing every month in cases requiring prolonged root canal therapy.

**Storage:**

- Refrigeration is recommended. Store material between 2°C/36°F to 27°C/80°F. Do not freeze. See outer package for expiry date.
- Do not expose to elevated temperatures or intense light.
- Keep syringe capped when not in use.

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