ES

Descripción:

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EN

Description:

Alpha-dent® Band Adhesive is a light cured, single paste resin-based releasing band adhesive. **Alpha-dent**® Band Adhesive requires no mixing; resulting in reducing adhesive waste, consistent adhesive performance and no working time constraints. **Alpha-dent**® Band Adhesive's distinctive blue color allows you to easily see the adhesive during banding, making clean up easy.

Indications for Use:

Alpha-dent® Band Adhesive is a light cured orthodontic adhesive designed for banding orthodontic appliances to teeth.

Contraindications:

Alpha-dent® Band Adhesive is contraindicated in patients with a history of severe allergic reaction to methacrylate resins.

General Precautions:

- U.S. Federal Law restricts this device to sale by or on the order of a dentist.
- Review instructions and SDS (www.dentaltech.com/resource) prior to beginning treatment. Use only as directed.
- Do not use in the presence of eugenol based materials as complete setting may not occur.
- Do not expose to elevated temperatures or intense light.
- Provide suitable moisture control to isolate area and to prevent moisture contamination during setting.

Adverse Reaction:

Allergic or pre-existing skin, eye and respiratory conditions may be aggravated by exposure to this product.

Instructions for Use:**Tooth Preparation**

1. Thoroughly clean teeth to be banded with **Alpha-Pro**® No Added Color/Flavor/Fluoride Proply Paste. Proply paste which contains oil or fluoride should not be used as it may inhibit the banding.
2. Rinse teeth thoroughly with water; dry with oil and moisture-free compressed air.

Band Adhesive Application

1. For proper fit, trial fit each band on the patient. After fit has been verified, remove band and clean.
2. Hold the band between the thumb and forefinger. Apply a bead of **Alpha-dent**® Band Adhesive to the inside of the band. Cover the entire inside of the band by spreading the paste with a small spatula or other suitable instrument.
3. Set band on tooth, the excess **Alpha-dent**® Band Adhesive can be removed with an explorer or plastic instrument.
4. Using a light-curing unit (minimum output of 600 mW/cm²) cure the material for at least 20 seconds. The light-curing unit tip should be placed in close proximity to the occlusal surface of the tooth being banded.
5. After treatment is complete, remove bands in the usual manner.

Storage:

- Recommended storage conditions are between 2°C/36°F to 27°C/80°F, bringing to room temperature before using (23°C/73°F). Do not freeze. See outer package for expiry date.
- Do not expose to elevated temperatures or intense light.
- Keep container closed when not in use.

DENTAL
TECHNOLOGIES

alpha-dent®
BAND ADHESIVE
LIGHT CURE

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