

TECHNICAL DATA SHEET

Product:	Alpha-CEM [®] DC Dual Cure Resin-Based Dental Luting Material		
Product Description:	Alpha-Cem [®] DC is a Dual Cure Resin-Based Luting Material containing 41% by volume of a blend of inorganic radio-opaque fillers with particle range from 0.04 to 2.2 microns possessing unsurpassed physical properties. The organic matrix oligomer is based on Bis-GMA. It is specially formulated for permanent cementation of esthetic ceramic/porcelain, etched metal resin bonded, or composite fixed prosthesis.		
Indications for Use:	Alpha-Cem [®] DC is recommended for cementation of implant prosthesis, crowns and bridges, inlays, Maryland Bridges, veneers, posts, pins and periodontal splinting.		
Appearance:	A two-paste system, available in highly translucent shade that adapts well to the restoration of the tooth structure. Base: Soft beige paste / Catalyst: Soft white paste.		
Curing Mechanism:	Dual cured by mixing catalyst and base together and applying an external energy activated Dental Blue Light with minimum output 600 mW/cm ² and wavelength of 400 to 500 nanometers. Self- cured by mixing catalyst and base together.		
Shelf-Life:	30 months		
Filler Type:	Barium glass, glass ionomer, quartz and silica.		
	Filler % by Volume:	Base:	41 %
		Catalyst:	45 %
	Filler Particle Range, μ:	Base:	0.02 – 2.0
		Catalyst:	0.01 – 5.0
	Filler Average Particle Size, μ:	Base:	1.6
		Catalyst:	4.7
Main Composition:	Base:	% w/w	
	Inorganic fillers	50 – 70	
	Bis-GMA	15 – 35	
	Methacrylate monomers	5 – 15	
	UV Absorber	< 1	
	Accelerator	< 1	
	Colorant	< 1	
	Photoinitiator	< 1	
	Catalyst:	% w/w	
	Inorganic fillers	50 – 70	
	Methacrylate monomers	10 – 30	
	BIS-GMA	10 – 20	
	Dessicant	< 5	
	Initiator	< 1	
	Preservative	< 1	

DENTAL TECHNOLOGIES, INC.

6901 N. HAMLIN AVENUE
LINCOLNWOOD, ILLINOIS 60712 USA

1.800.835.0885 TOLL FREE [US/CANADA]

1.847.677.5500 PHONE

1.847.677.5502 FAX

Physical /Mechanical Properties: (Typical Values)

Setting Time @ (37 ± 1) °C, minutes:	4
Water Sorption, µg / mm ³ :	27
Water Solubility, µg / mm ³ :	2.3
Film Thickness, µm:	20
Flexural Strength, MPa :	124
Compressive Strength, MPA:	220
Diametral Strength, MPA:	43
Working Time @ (23 ± 2) °C, seconds:	120 (minimum)

DENTAL TECHNOLOGIES, INC.

6901 N. HAMLIN AVENUE
LINCOLNWOOD, ILLINOIS 60712 USA

1.800.835.0885 TOLL FREE [US/CANADA]
1.847.677.5500 PHONE
1.847.677.5502 FAX