

TRAY ADHESIVE



Description:

Alpha-dent® Tray Adhesive is used to obtain the most accurate impressions possible by adhering impression materials to trays. Since impression materials do not have adhesive properties, it is advisable to use **Alpha-dent®** Tray Adhesive on all trays prior to taking an impression.

Indications for Use:

Bonds alginate and hydrocolloid impression materials to metal or plastic trays.

Contraindications:

Not recommended for use with elastomeric impression materials.

Warning:



- Avoid contact with eyes or skin. In case of contact with eyes, flush with copious amounts of water for at least 15 minutes. Seek immediate medical attention.
- Vapor is harmful. Use only in a well ventilated area. If inhaled, move to fresh air.
- Do not apply directly to teeth, skin or oral tissues.
- **Alpha-dent®** Tray Adhesive is flammable. Keep away from heat, sparks or open flame.
- This product contains a chemical known to the State of California to cause birth defects or other reproductive harm.

General Precautions:

- U.S. Federal Law restricts this device to sale by or on the order of a dentist.
- Review instructions and SDS (www.dentaltech.com/resource) prior to beginning treatment. Use only as directed.

Adverse Reaction:

Allergic or pre-existing skin, eye and respiratory conditions may be aggravated by exposure to this product.

Instructions for Use:

1. Brush a thin layer of **Alpha-dent®** Tray Adhesive on the inside surface of the tray and slightly over the peripheral border. Allow to dry for approximately 3 minutes.
2. Place the impression material in the tray while **Alpha-dent®** Tray Adhesive is still tacky.
3. Take the impression as directed for the impression material.
4. After the impression material has set, it will remain attached to the tray, thus helping to achieve a more accurate impression.
5. Clean tray with isopropyl alcohol.

Storage:

1. Recommended storage conditions are between 2°C/36°F to 27°C/80°F, bringing to room temperature before using (22°C/72°F). See outer package for expiry date.
2. Do not expose to elevated temperatures or sources of ignition.
3. Keep containers tightly closed when not in use.



**DENTAL
TECHNOLOGIES**

6901 N. HAMLIN AVENUE
LINCOLNWOOD, ILLINOIS 60712 USA

www.dentaltech.com

EC

REP

EMERGO EUROPE

Prinsessegracht 20
2514 AP, The Hague
The Netherlands