




DENTAL  
TECHNOLOGIES

www.dentaltech.com

# ALPHA-PRO®

PREVENTIVES

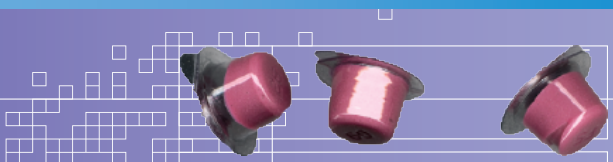

## PROPHYLAXIS PASTE

Net Wt. 400 GRAMS  
(200 SINGLE USE CUPS)

- MEDIUM, No Fluoride, No Flavor / REF 560.I30I.002
- MEDIUM MINT / REF 560.0204.002
- MEDIUM CHERRY / REF 560.0304.002
- COARSE MINT / REF 560.I204.002
- COARSE CHERRY / REF 560.0604.002



**DENTAL  
TECHNOLOGIES**

6901 N. HAMLIN AVENUE  
LINCOLNWOOD, ILLINOIS 60712 USA

**ALPHA-PRO®** is a blend of polishing and cleaning agents designed for professional use during standard dental/orthodontic hygiene procedures. Each unit dose cup contains enough paste for a single prophylaxis treatment.

### Indications for use:

For cleaning and polishing procedures as part of a professionally administered dental/orthodontic prophylaxis treatment. If product contains fluoride, it is not considered a substitute for other forms of fluoride treatment.

### Instructions for use:

1. Using a prophy angle, fill the cup by pressing the face of the cup into the paste while running the angle at a low speed.
2. Maintain a quick and light touch so the edges of the cup flair slightly while continuing moving from the tooth. Sufficient paste should be maintained on the cup to avoid

overheating of the tooth surface.

3. Evacuation of saliva is recommended to avoid dilution and swallowing of the paste.

### Storage:

Do not store at temperature above 25°C/77°F. Keep out of direct sunlight. See outer package for expiration date.

### Warnings:



- Paste should be expectorated and not swallowed.
- Product contains 1.23% Sodium Fluoride, unless noted, nausea can result from excessive ingestion of fluoride.
- Keep out of reach of children.



EMERGO EUROPE  
Prinsessegracht 20  
2514 AP, The Hague  
The Netherlands



CAUTION: U.S. Federal Law  
restricts this device to sale by or  
on the order of a dentist.

MADE IN U.S.A. PM-4266/REV005