



DENTAL  
TECHNOLOGIES

www.dentaltech.com

# ★ ALPHA-PRO® KID'S PACK ★



Sweet Cotton Candy    Chocorific Chocolate    Berry-Licious    Watermelon Wonder

**ALPHA-PRO® KID'S PACK** is a blend of polishing and cleaning agents designed for professional use during standard dental hygiene procedures. Each unit dose cup contains enough paste for a single prophylaxis treatment.

**Indications for use:**

For cleaning and polishing procedures as part of a professionally administered dental prophylaxis treatment. Product is not considered a substitute for other forms of fluoride treatment.

**Instructions for use:**

1. Using a prophylaxis angle, fill the cup by pressing the face of the cup into the paste while running the angle at a low speed.
2. Maintain a quick and light touch so the edges of the cup flair slightly while continuing moving from the tooth. Sufficient paste should be maintained on the cup to avoid overheating of the tooth surface.
3. Evacuation of saliva is recommended to avoid dilution and swallowing of the paste.
4. Rinse full mouth with water spray.

**Storage:**

Do not store at temperature above 25°C/77°F. Keep out of direct sunlight. See outer package for expiration date.

**Warnings:**

- Paste should be expectorated and not swallowed.
- Product contains 1.23% Fluoride Ion, nausea can result from excessive ingestion of fluoride.
- Keep out of reach of children.



DENTAL TECHNOLOGIES 6901 N. Hamlin Ave., Lincolnwood, IL 60712 U.S.A.



CAUTION: U.S. Federal Law restricts this device to sale by or on the order of a dentist.



EMERGO EUROPE  
Prinseweg 23  
2513 BQ, The Hague  
The Netherlands



PM-4705/REV004



\*+D7355600000217

★ 200 Fun Stickers Included ★ Combo Pack 50 of each Yummy Flavor ★

PROPHYLAXIS PASTE • NET WT. 400 GRAMS ( 200 SINGLE USE CUPS) • MEDIUM GRIT • REF 560.0000.002